



ic LAVAGE

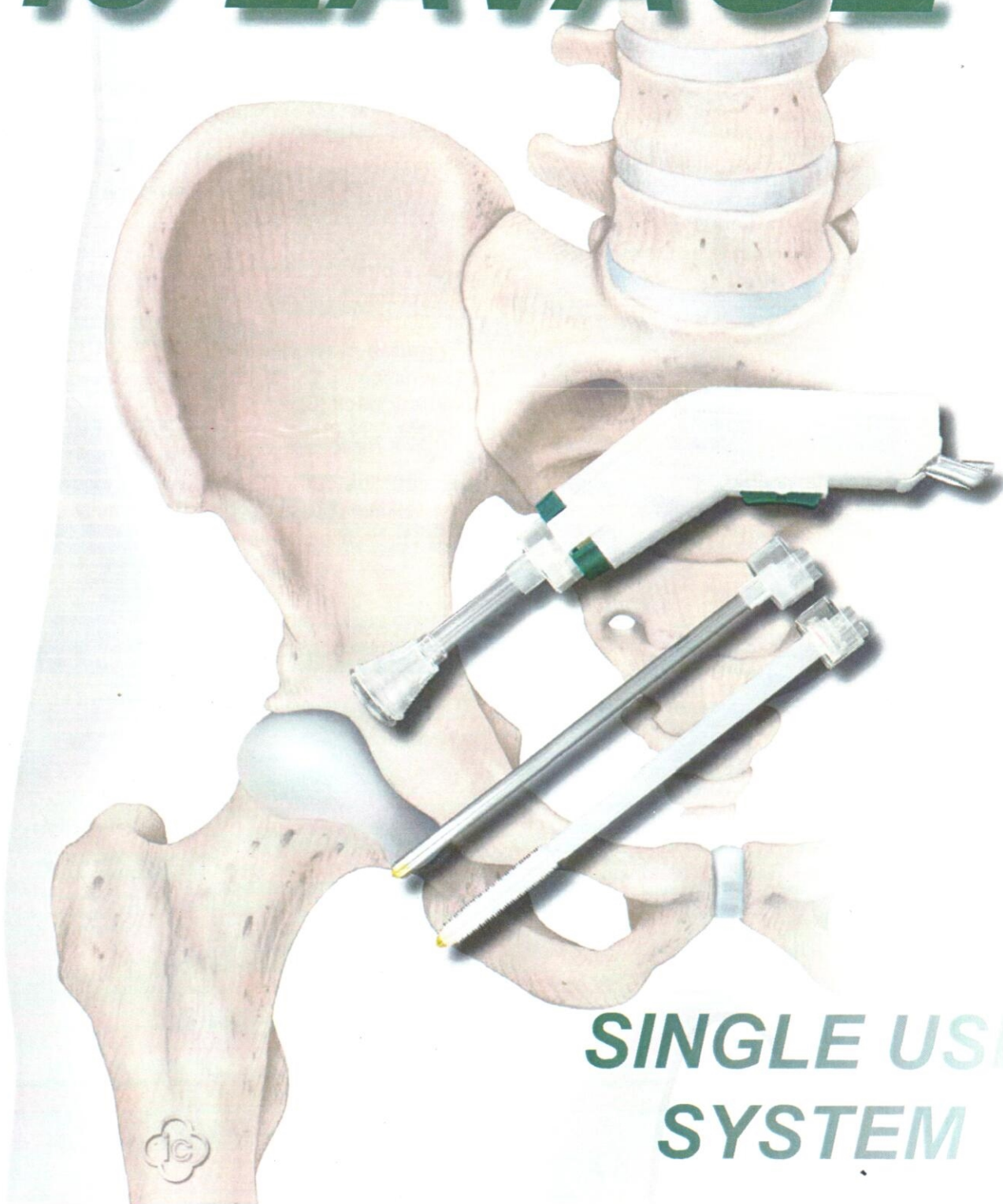

**SINGLE USE
SYSTEM**

The ic-lavage system combines convenience and power for a gentle, efficient cleaning and the removal of loose bone -, tissue - and blood particulates. At the same time the system assures an efficient lavation with controlled pressure.

The disposable system is delivered in a double sterile packaging and includes a lightweight hand piece with two nozzles. It is battery powered and can be controlled with a fingertip trigger with the options of powerful cleaning or smooth flush. The clamp 3.1. can regulate the liquid flow in the irrigation pipe and waste pipe. By sucking and flushing concurrently, debris will be removed without flooding the operative site.

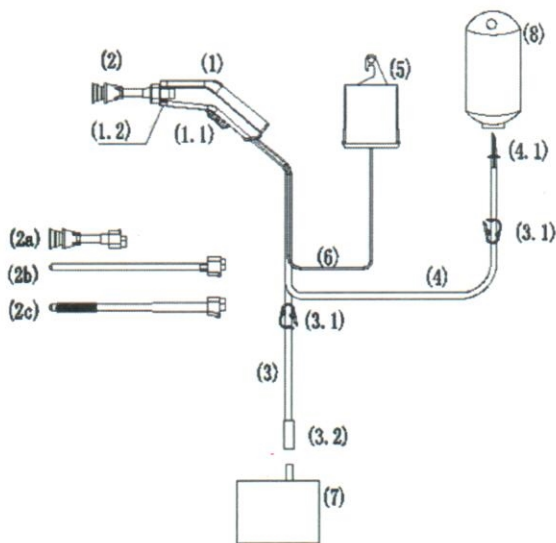

tool handle (1), contains:

DC motor, compression pumps, trigger (1.1), nozzle lock (1.2), enclosure.

jet nozzle (2), contains:

fan spray nozzle (2a), femoral nozzle (long) (2b) femoral brush (long) (2c).








waste pipe (3), contains: clamp (3.1), connecting pipe (3.2).

irrigation pipe (4), contains: clamp (3.1), Luer-fitting (4.1).

battery pack (5).

power wire (6).


The suction drain (7) and the bag for the flushing solution (8) are not included in delivery. They have to be allocated by operator.

-  LIGHTWEIGHT CONSTRUCTION
-  SLIM, ERGONOMIC DESIGN
-  COMFORTABLE IN HANDLING AND USING
-  SECURE TIP LOCKING MECHANISM
-  CARES FOR A SECURE FIXATION OF THE ATTACHMENT
-  FINGERTIP-CONTROL
-  EASY SWITCHING BETWEEN POWERFUL CLEANING AND GENTLE LAVAGE

FULL SET

REF: 00678000 ic Lavage:

The full set includes a tool holder, a short nozzle with a splash guard shield and an integrated suction as well as a femur nozzle with lavage and suction (packing unit: 10 pcs.):

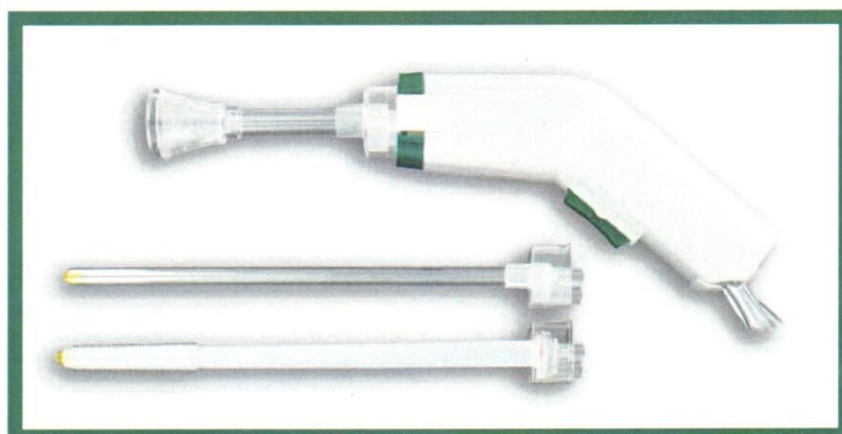

OPTIONAL ACCESSORY

REF: 00678002 ic Lavage:

Long femoral brush (dimension: 13,5mm x 250mm) with lavage and suction (packing unit: 5 pcs.):



The set is directly ready for use. Clip the nozzle on, link the connecting joser with the jetting liquid and the suction and hang the battery to the infusion holder.

TECHNICAL SPECIFICATIONS:

weight tool holder	< 0,9 kg
power	10 x 1,5 V battery
volume	< 75db (A)
flow rate	> 0,9 L/min.
max. range	> 2,5 m



implantcast

implantcast GmbH
Lüneburger Schanze 26
21614 Buxtehude
Germany
phone: +49 4161 744-0
fax: +49 4161 744-200
e-mail: info@implantcast.de
internet: www.implantcast.de

Your local distributor

MANUFACTURER

GUANGZHOU CLEAN MEDICAL PRODUCTS MANUFACTURING CORP
NO. 2 GUANGJING ROAD
CONGCHUA DEVELOPMENT DISTRICT
GUANGZHOU
CHINA
TEL: +86-20-3785030
FAX: +86-20-3785032

DISTRIBUTION

IMPLANTCAST GMBH
LÜNEBURGER SCHANZE 26
D-21614 BUXTEHUDE
PHONE: +49 4161 744-0
FAX: +49 4161 744-200
E-MAIL: info@implantcast.de
INTERNET: WWW.IMPLANTCAST.DE

